

SUBMITTING CANDIDA ISOLATES

BECOME A SUBMITTER


Email the OHA Healthcare-associated Infections (HAI) Program to get started.

HAI@odhsoha.oregon.gov

PREP THE PACKAGE

Isolates should be sub-cultured or cultured on a non-inhibitory, selective agar slant (e.g. Sabouraud's Dextrose or other appropriate media).

Each specimen needs a test form. Complete all the required sections. The isolate must be labeled with FIRST and LAST NAME & DATE OF BIRTH. It must match the test form.

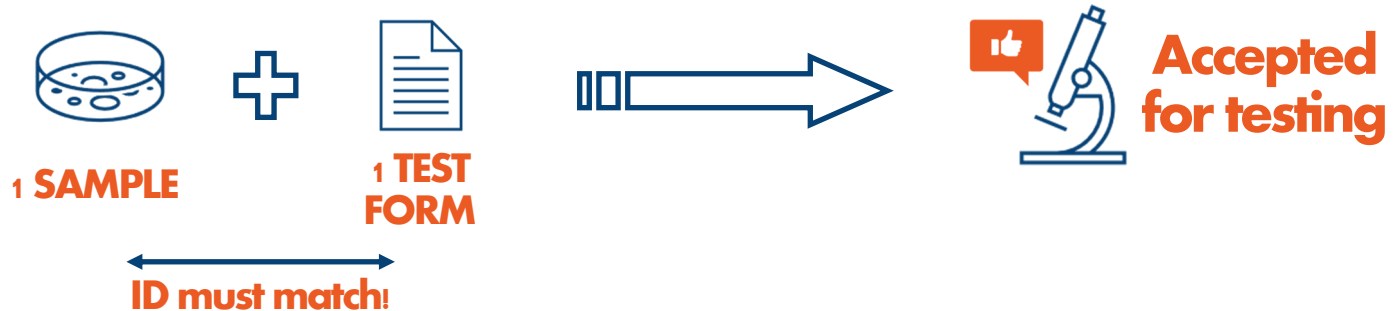

SHIP IT

Shipping is free using pre-paid FedEx labels. Contact the OHA HAI Program to request labels. Isolates should ship within 7 days of collection, at room temp or with refrigeration. Testing occurs Monday—Friday.

WA State Public Health Lab
Attn: ARLN
1610 NE 150th Street
Shoreline, WA 98155



healthoregon.org/HAI



OREGON
HEALTH
AUTHORITY